# DISCOVER IRIS

# WHY IRIS ?

- LED TECHNOLOGY
- SCIALYTIC EFFECT

- LIGHT INTENSITY  
ADJUSTMENT UP TO  
**50.000 LUX**
- COLOUR  
TEMPERATURE  
ADJUSTMENT

- SURGERY MODE
- COMPO SAFE MODE
- COLOUR MATCHING  
MODE

**KEYBOARD**

# LIGHT INTENSITY ADJUSTMENT

**HOW MUCH LIGHT ?**

**35.000 LUX**


**BEST DETAIL PERCEPTION  
PREVENTING EYESTRAIN**

**50.000 LUX**

**FOR SPECIFIC NEEDS DURING  
SHORT PERIODS OF TIME**

# COLOUR TEMPERATURE ADJUSTMENT

**CHOOSE**  
**THE RIGHT COLOUR TEMPERATURE FOR EVERY TREATMENT.**

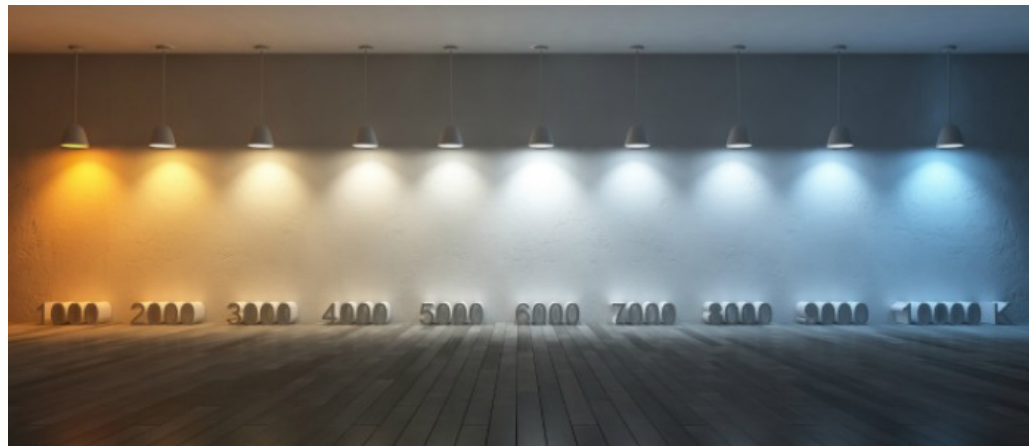- COLD LIGHT (OPTIMISES THE COLOUR CONTRAST ON SOFT TISSUES) --> SURGERY
- 5.500 K (SUN LIGHT) --> COLOUR MATCHING

# SCIALYTIC EFFECT

**OVERLAPPING LIGHT RAYS**  
**AVOID SHADOWS PROJECTION**  
**ON THE OPERATING FIELD**


# RISK FROM WRONG LIGHTING

## INADEQUATE COLOUR TEMPERATURE

- POSSIBLE MISTAKES IN COLOUR MATCHING PROCEDURES
- MAKES IT DIFFICULT TO DISTINGUISH SHADES OF RED OF SOFT TISSUES

## LACK OF SCIALYTIC EFFECT

- BLURRED PERCEPTION OF THE OPERATING FIELD
- INNATURAL POSTURE WITH POSSIBLE CERVICAL AND LUMBAR DISEASES

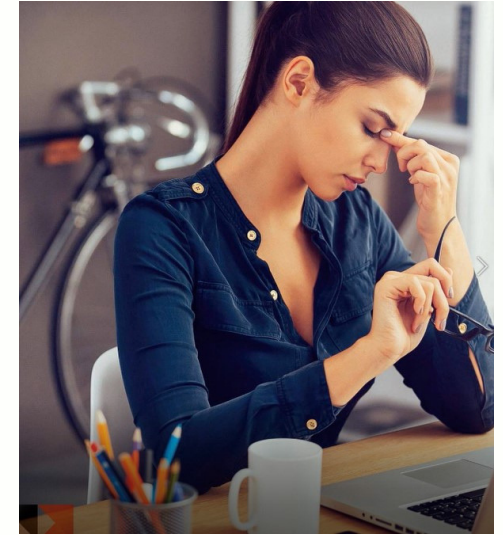## WRONG LIGHT INTENSITY

- BLURRED SIGHT: DETAILS ARE I

# ACCESSORIES

- **MIRROR KIT**

IT IMPROVES THE COMMUNICATION WITH PATIENTS

OPTIONAL

- **WHITENING LIGHT  
"CORE WHITE"**

OPTIONAL

# EASY HYGIENE

**\* EASY CLEANABLE FRONT COVER**

**\* PARABOLES ARE PROTECTED AND  
NEED NO CLEANING**

**\* REMOVABLE HANDLE COVERS**  
**- AUTOCLAVABLE 15 MINUTES AT  
135° (ISO 11134)-**  
**- 2 PAIRS PROVIDED WITH EVERY  
LIGHT**

# TECHNICAL FEATURES

## IRIS

- SUPPLY VOLTAGE: 12 - 24 VAC 50-60 HZ
- MAXIMUM POWER ABSORBED: 50 VA
- ILLUMINATED SPOT DIMENSION: 80 X 200 MM
- LIGHT INTENSITY (5 STEPS): FROM 8.000 TO 50.000 LUX
- COLOUR TEMPERATURE (5 STEPS): FROM 4200 K TO 6000 K
- OPERATING LENGTH: 70 CM
- LIGHT SOURCE: 8 LEDS
- GUARANTEE: 24 MONTHS

## CORE WHITE WHITENING LIGHT

- WEIGHT: 800 G
- POWER SUPPLY: 17-24 VAC
- MAXIMUM POWER CONSUMPTION: 30 VA
- CURRENT CONSUMPTION: 1.5 A
- LIGHT PATTERN: 80X20 MM
- LIGHT SOURCE: 4 LED (3.5 W CAD.)
- EMISSION SPECTRUM: 420-480 NM
- MAXIMUM LIGHT POWER EMITTED: 2000MW
- MAXIMUM DISTANCE FROM PATIENT: 10 CM

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LIGHTS THAT  
GUARANTEE OPTIMUM VISUAL  
COMFORT, COLOUR PERCEPTION,  
AND PREVENT EYE-STRAIN.

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